


## SENIOR POMMEL WALKER Adjustment Chart


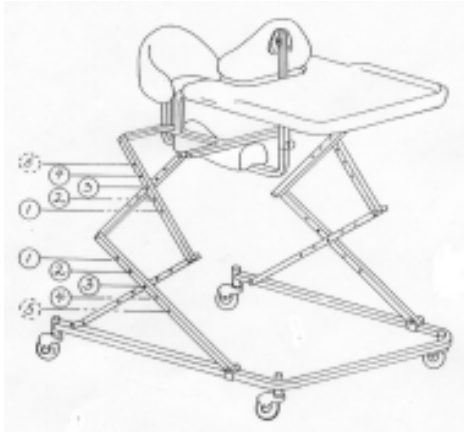


### To change to a new position:

- Lay the walker on its side on the floor.
- Remove the link bolts from both sides; extend the walker to the desired size and decide which hole the bolts will go into.
- Turn the walker onto the other side and adjust accordingly.
- Replace the bolts. Mark the holes with tape or a felt pen.

Measurement in INCHES

Position Number	1	2	3	4	5
Pommel Height (B)	30 to 36	27 to 33	24 to 30	21 to 27	18 to 24
Tray Height	47.5	44.5	41.5	38.5	35.5
Chest Pad Height (front) (A)	52.5 to 59	49.5 to 56	46.5 to 53	43.5 to 50	40.5 to 47

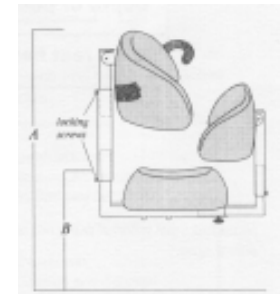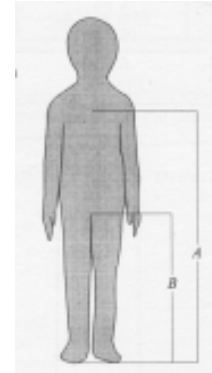


A is measurement from mid-sternum to floor

B is measurement from crotch to floor

### To adjust the pommel and chest pad height:

1. Loosen the locking screws or knobs on the front of the pommel stem (see figure at left)
2. Move the pommel and chest pad up or down to the required position.
3. Tighten the screws and knobs securely.

## SENIOR POMMEL WALKER Adjustment Chart


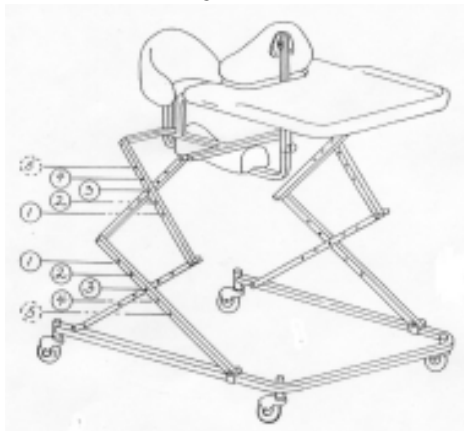


### To change to a new position:

- Lay the walker on its side on the floor.
- Remove the link bolts from both sides; extend the walker to the desired size and decide which hole the bolts will go into.
- Turn the walker onto the other side and adjust accordingly.
- Replace the nuts. Mark the holes with tape or a felt pen.

Measurement in INCHES

Position Number	1	2	3	4	5
Pommel Height (B)	30 to 36	27 to 33	24 to 30	21 to 27	18 to 24
Tray Height	47.5	44.5	41.5	38.5	35.5
Chest Pad Height (front) (A)	52.5 to 59	49.5 to 56	46.5 to 53	43.5 to 50	40.5 to 47

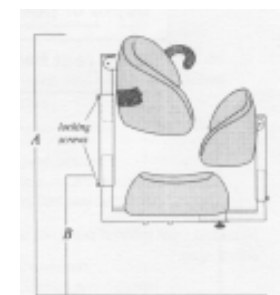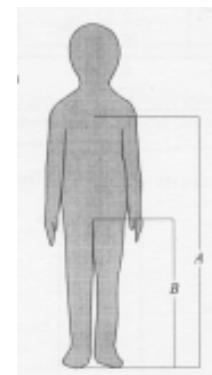

A is measurement from mid-sternum to floor

B is measurement from crotch to floor

### To adjust the pommel and chest pad height:

1. Loosen the locking screws or knobs on the front of the pommel stem (see figure at left)
2. Move the pommel and chest pad up or down to the required position.
3. Tighten the screws and knobs securely.

The pommel walker is designed as a walking aid for children who are unable to walk without support.

It is recommended that this product be used under the direction of a physiotherapist or occupational therapist.

**Please save these instructions for future use.**

MANUFACTURED BY:



633 Wellington Crescent  
Winnipeg, MB R3M 0A8  
Phone (204) 452-4311  
Fax: (204) 477-5547

The pommel walker is designed as a walking aid for children who are unable to walk without support.

It is recommended that this product be used under the direction of a physiotherapist or occupational therapist.

**Please save these instructions for future use.**

MANUFACTURED BY:



633 Wellington Crescent  
Winnipeg, MB R3M 0A8  
Phone (204) 452-4311  
Fax: (204) 477-5547

## WARNINGS

- Ensure the environment is safe before using.
- Never leave child unattended.
- Do not use as a transportation device.
- Inspect walker daily for loose, broken or missing parts.
- DO NOT USE if there are loose, broken or missing parts.
- DO NOT USE if the product feels unsafe.
- Contact service provider or qualified service technician when repairs or service are required.
- Failure to follow these instructions could result in serious injury.

## WARNINGS

- Ensure the environment is safe before using.
- Never leave child unattended.
- Do not use as a transportation device.
- Inspect walker daily for loose, broken or missing parts.
- DO NOT USE if there are loose, broken or missing parts.
- DO NOT USE if the product feels unsafe.
- Contact service provider or qualified service technician when repairs or service are required.
- Failure to follow these instructions could result in serious injury.

## SENIOR POMMEL WALKER



## USERS GUIDE

## SENIOR POMMEL WALKER



## USERS GUIDE